

Charging Dock User Manual




D1013315 A

1 Package Contents

Package Contents:

- 1 x Charging dock
- 1 x USB charging adapter
- 1 x USB-C charging cable
- 1 x User manual

2 Features

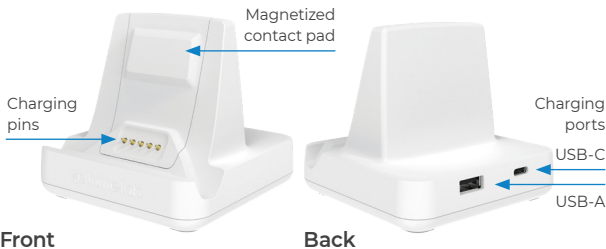

3 Setup

3.1 Initial Setup

1. Place charging dock on a flat, stable surface.
2. Connect the USB charging cable to the USB charging adapter.
3. Plug the USB charging adapter into an approved socket and turn on.
4. Insert the end of the USB-C charging cable into the charging dock USB-C port.
5. Place the ellume-lab into the charging dock and ensure charging pins are aligned. The charging symbol will appear in the top left corner of the ellume-lab screen.

3.2 Charging Pin Alignment







4 Maintenance and Cleaning


Routinely clean the charging dock by wiping each surface for at least 30 seconds using standard disinfection wipes soaked in min. 70% isopropyl alcohol, e.g. Clinell Wipes.

- × Do not clean the charging dock with unapproved products or solutions.
- × Do not open or disassemble the charging dock.
- × Do not immerse the ellume-lab charging dock in water or cleaning solutions.

5 Safety

5.1 Universal Labels and Symbols

	Manufacturer
	Date of manufacture
	Consult instructions for use
	Temperature limitation
REF	Catalogue number

	CAUTION Consult accompanying documents.
	WARNING Indicates a hazardous situation, which if not avoided, could result in injury to the operator (e.g. electrical shock)
LOT	Lot number

5.2 Warnings & Safety Precautions

The ellume·lab charging dock is designed to ensure safe and reliable operation when used in accordance with this user manual. If the charging dock is used in ways other than specified in this manual, safety of the operator and operation of the ellume·lab may be compromised. All warnings and precautions must be followed to avoid personal injury or damage:

- Remove the ellume·lab from the charging dock before performing a test.
- The ellume·lab charging dock must be used with a standard electrical socket, provided charging cable and region-specific adapter.
- Using non-approved cables and accessories will impact charging dock function and void warranty.
- Incorrect use of the ellume·lab charging dock and charging cable, or the USB charging adapter may damage these items.

6 Technical Specifications

Dimensions	3.39 x 3.39 x 3.05 inches
Weight	16.03 ounces
Power Supply Input	5V $\overline{=}$ 2A
USB-A Output	5V $\overline{=}$ 0.5A
Operating environment	59°F – 95°F [+15°C, +35°C], 20% - 90% RH (non-condensing)
Shipping	-4°F – 131°F [-20°C, +55°C], 20% - 90% RH (non-condensing)
Storage	59°F - 95°F [+15°C, +35°C]

7 Ellume Contact Information

Contact Ellume customer support:

Tel: 1-888-885-6121

Fax: 1-661-335-6330

Email: support@ellumelab.com

Website: www.ellumelab.com

8 Additional Information

For customer support, call 1-888-885-6121 or visit www.ellumelab.com.

Test system problems may also be reported to the FDA through the MedWatch medical products safety reporting program (phone: 1-800-FDA (332)-1088; fax: 1-800-FDA (332)-0178; online: <http://www.fda.gov/medwatch>).

Information correct as of December 21 2020.



Ellume Limited

57 Didsbury St, East Brisbane,

Qld 4169, Australia



ellumehealth.com