


# Quick Start Guide

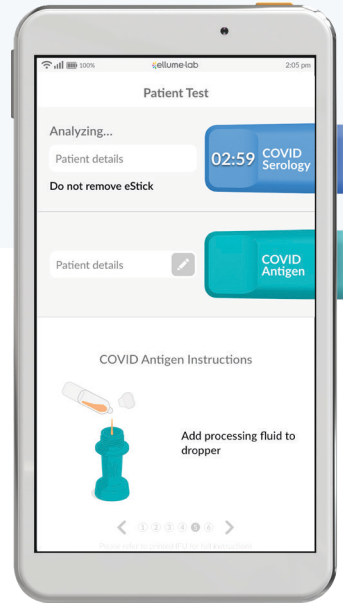

## Getting Started

- 1 Activate your account via the link in your welcome email  
*\*Ensure to remember your password*
- 2 Fully charge the ellume-lab before initial use
- 3 Turn on by pressing and holding the power button for 2-3 seconds
- 4 Connect to a secure Wi-Fi network
- 5 Select User:

### Admin and Registered Users

- Select your account from the user list
- Enter your account password
- Create a PIN

**or**

### New User

- Tap the 'New User' button from the user list
- Fill in the Email Request form

*\* You will receive a welcome email once the ellume-lab administrator approves your request*

- 6 You are now ready to use the ellume-lab

## Important Information

- Completion of test specific training is required before performing a Patient Test or Control Test. Tap the Training icon on the Home Screen to begin.
- For test specific information, please refer to the Instructions for Use and Quick Reference Instructions in the test carton.

D1025921B



Please see the Package Insert (Instructions for Use) and User Manual for complete information regarding ellume-lab tests. This is not a complete Package Insert.

For more information, refer to Package Insert (Instructions for Use) and User Manual or contact Ellume customer support on 1-888-919-0779 or at support@ellumelab.com



# Getting to know your ellume·lab



- |                  |                     |                         |
|------------------|---------------------|-------------------------|
| 1 On/Off Button  | 5 Home Screen       | 9 Rear Camera           |
| 2 Charging Port  | 6 Control Test Icon | 10 eStick Port (Top)    |
| 3 Front Camera   | 7 Patient Test Icon | 11 eStick Port (Bottom) |
| 4 Volume Control | 8 Settings Icon     | 12 Speaker              |

## When used with Ellume's Emergency Use Authorized ellume·lab COVID Antigen test:

For *in vitro* diagnostic use only. For prescription use only. This product has not been FDA cleared or approved but has been authorized by FDA under an Emergency Use Authorization (EUA) for use by authorized laboratories; use by laboratories certified under the CLIA, 42 U.S.C. §263a, that meet the requirements to perform moderate complexity, high complexity, or waived tests. This test is authorized for use at the Point of Care (POC), i.e., in patient care settings operating under a CLIA Certificate of Waiver, Certificate of Compliance, or Certificate of Accreditation. The emergency use of this test product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of *in vitro* diagnostic tests for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Act, 21 U.S.C. § 360bbb-3(b) (1), unless the declaration is terminated or authorization is terminated or revoked sooner. The test product is authorized only for the detection of proteins from SARS-CoV-2, not for any other viruses or pathogens.